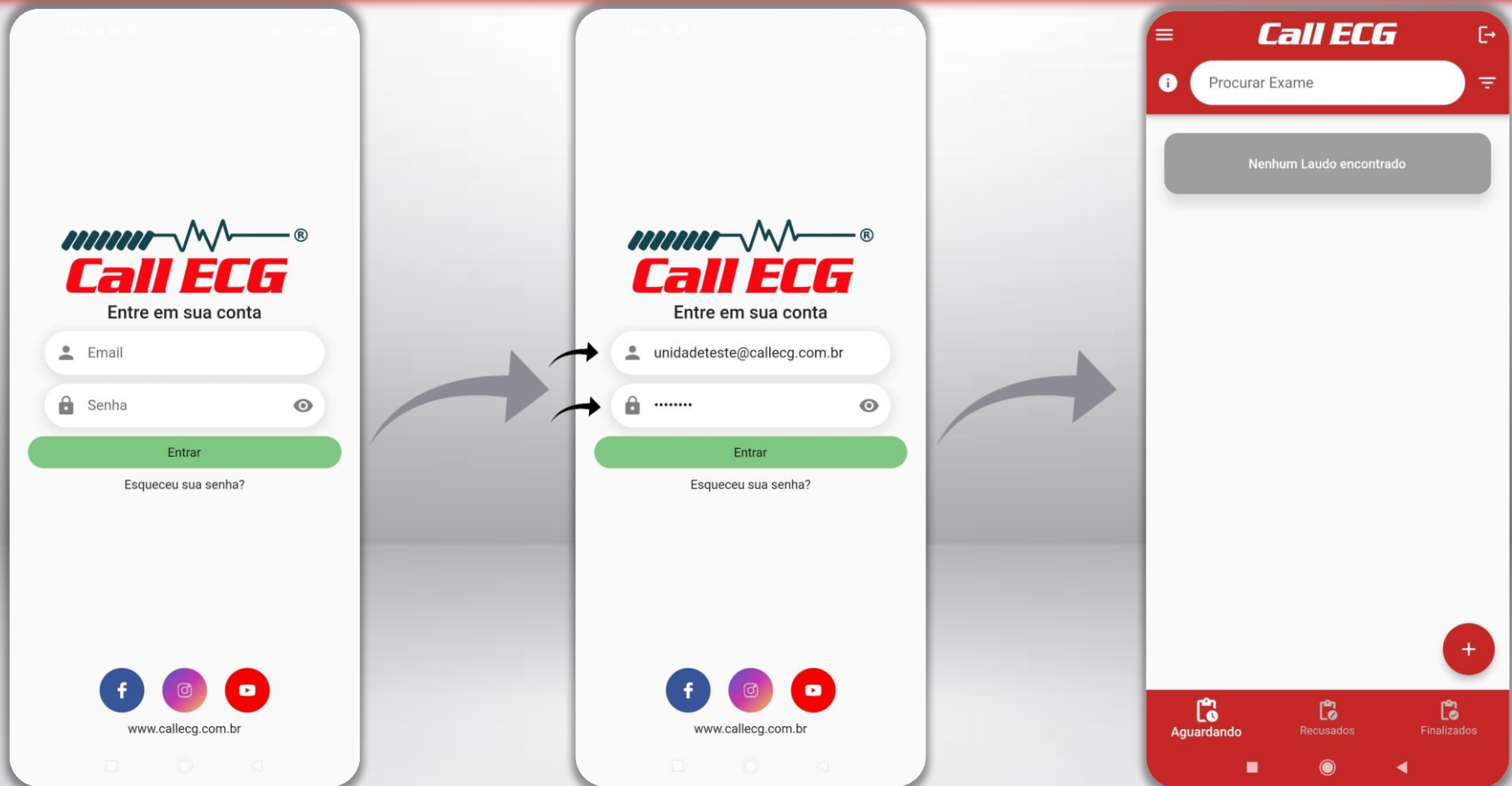
The background features a stylized illustration of a human heart with its major blood vessels, rendered in shades of red and purple. Overlaid on this are two horizontal ECG (heart rate) lines in a golden-brown color, one at the top and one at the bottom of the frame.

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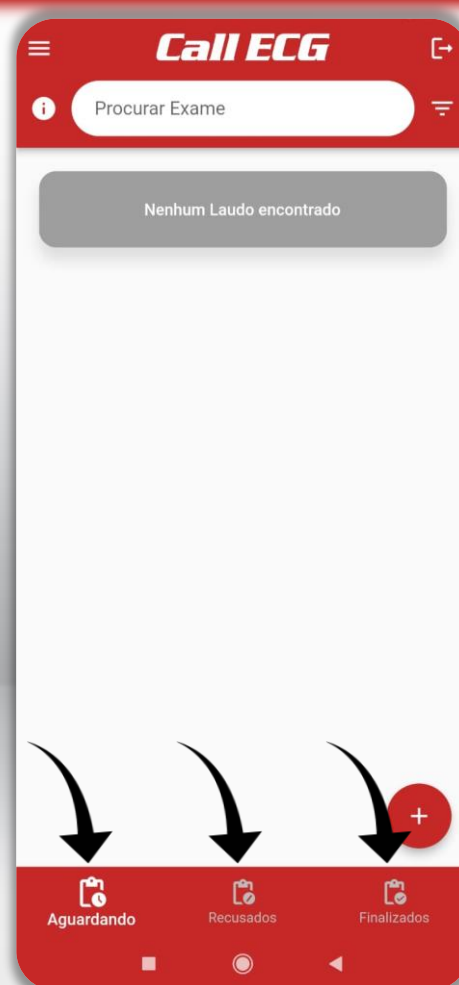
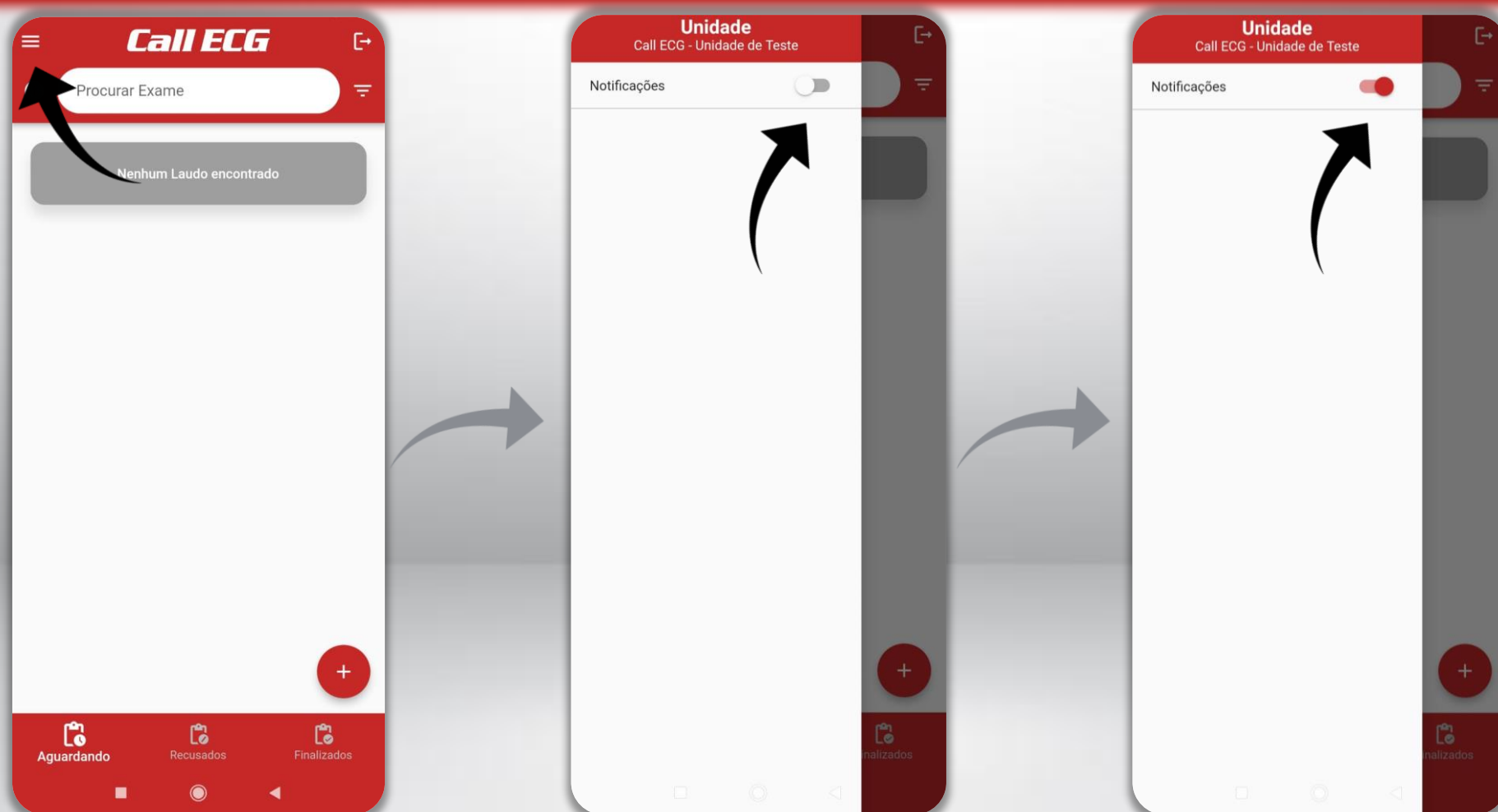
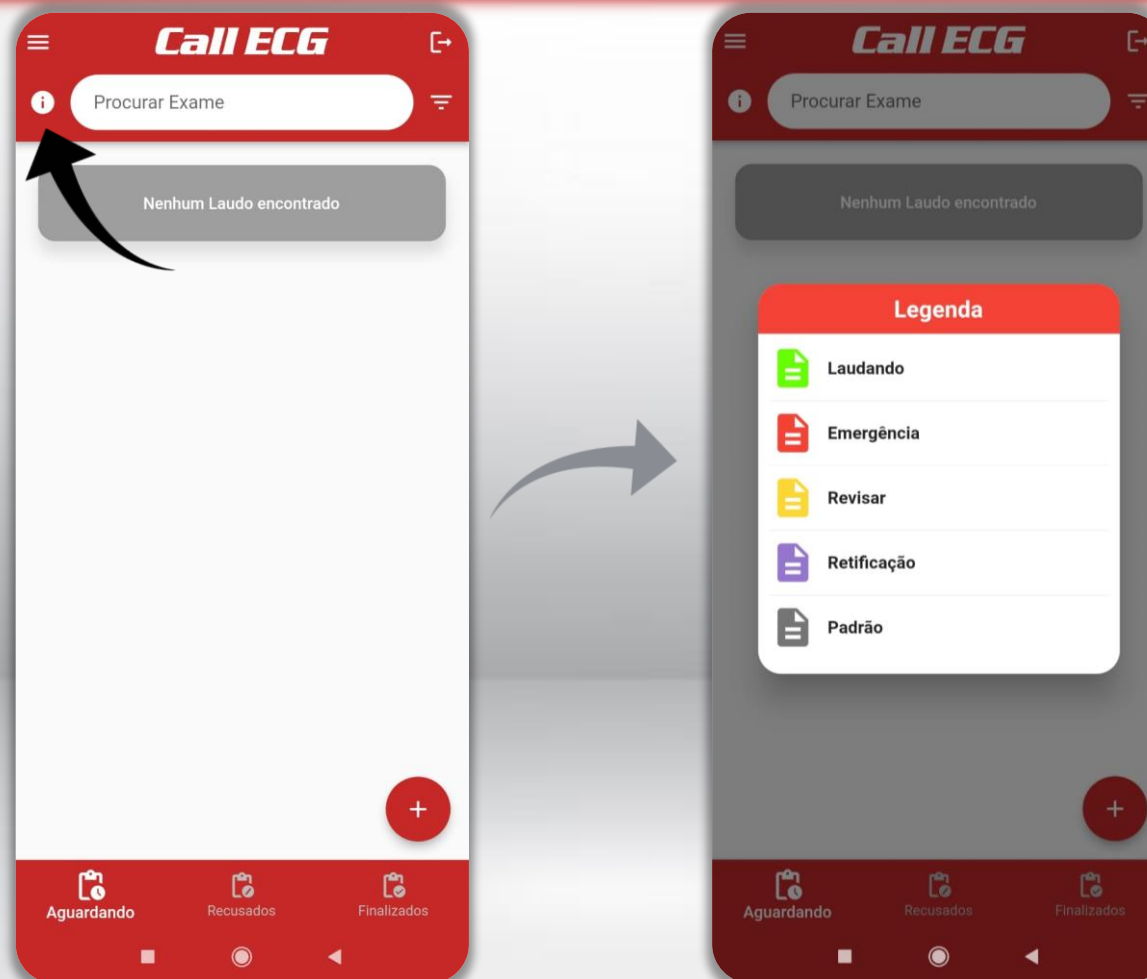
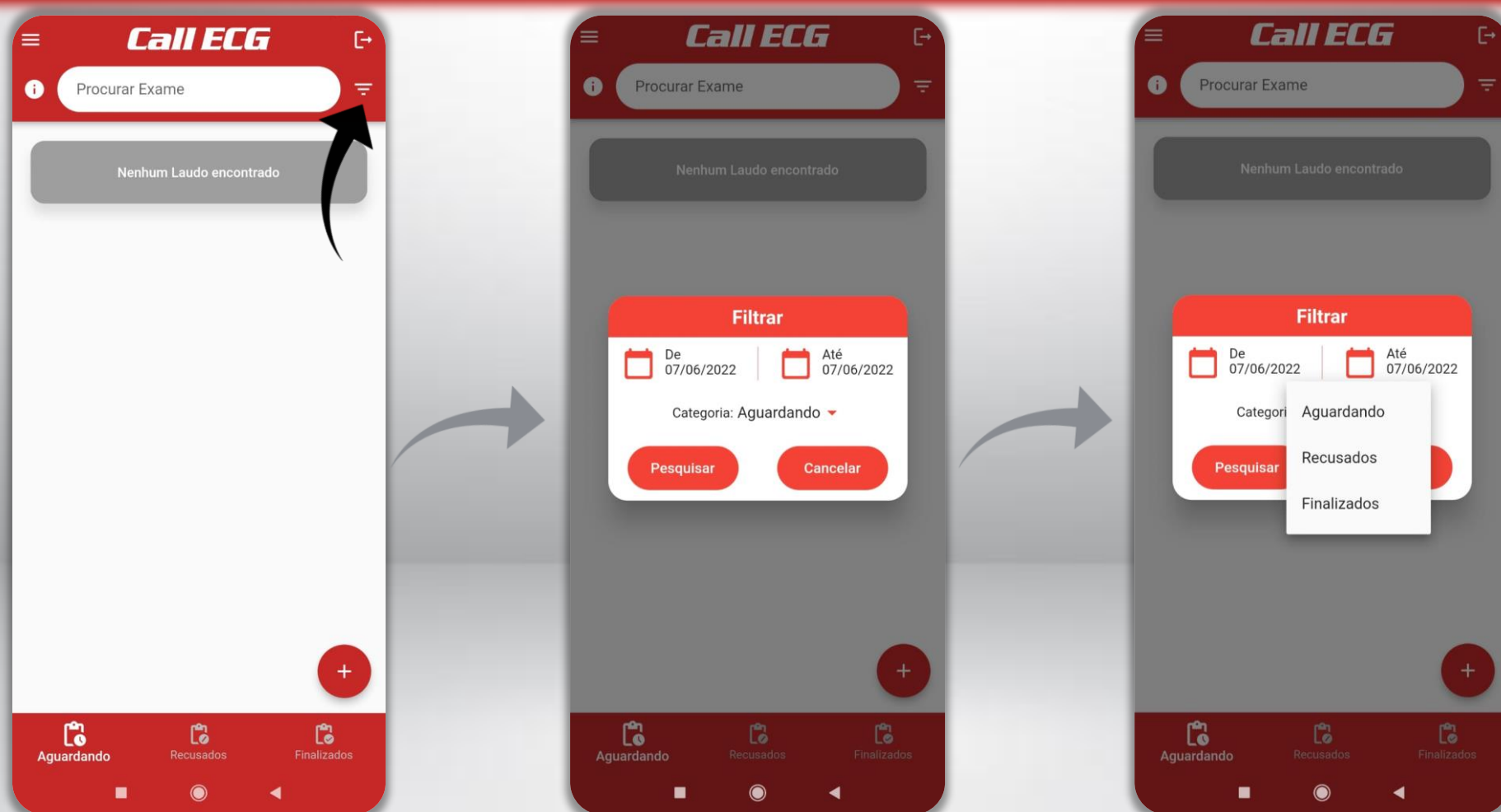
The image illustrates the process of adding a new exam in the Call ECG mobile application. It consists of three sequential screens:

- Screen 1 (Procurar Exame):** The main search screen. It features a search bar at the top with the text "Procurar Exame" and a red header with the "Call ECG" logo. A message "Nenhum Laudo encontrado" is displayed in the center. At the bottom right, there is a red circular button with a white "+" symbol, which is highlighted by a curved arrow.
- Screen 2 (Novo Exame):** The form for adding a new exam. It has a red header with a back arrow and the title "Novo Exame". The form includes several input fields and dropdown menus: "CPF *" (with a search icon), "Paciente *" (with a person icon), "Data de Aniversário *" (with a calendar icon), "Sexo *" (with a gender icon), "Prioridade *" (with a bar chart icon), "Usa Marcapasso? *" (with a heart icon), "Telefone *" (with a phone icon), and "Numero de Atendimento/..." (with a list icon). Below these fields are several buttons for symptoms: "DOR PRECORDIAL", "DISPNÉIA", "CEFALÉIA", "SEM MEDICAMENTO", "PRÉ OP.", "CHAGAS", "NÁUSEAS E/OU VÔMITOS", and "SUDORESE". A curved arrow points from the "+" button on the first screen to this form.
- Screen 3 (Novo Exame):** The same form as Screen 2, but with data entered. The fields are filled with: "CPF *" 123.456.789-01, "Paciente *" PACIENTE DE TESTE, "Data de Aniversário *" 10/10/1980, "Sexo *" Feminino, "Prioridade *" Eletivo, "Usa Marcapasso? *" Não, and "Telefone *" (00)00000-0000. The symptom buttons "DOR PRECORDIAL", "SEM MEDICAMENTO", and "SUDORESE" are highlighted in red. A curved arrow points from the form on Screen 2 to this screen.

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The screenshot shows a mobile application interface for creating a new exam. The form is titled 'Novo Exame' and contains the following fields and buttons, with numbered arrows indicating the sequence of input:

- 1** points to the 'CPF *' field.
- 2** points to the 'Paciente *' field.
- 3** points to the 'Data de Aniversário *' field.
- 4** points to the 'Usa Marcapasso? *' dropdown menu.
- 5** points to the 'Sexo *' dropdown menu.
- 6** points to the 'Telefone *' field.
- 7** points to the 'Prioridade *' dropdown menu.
- 8** points to the 'Numero de Atendimento/...' field.
- 9** points to the 'NÁUSEAS E/OU VÔMITOS' button.

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